
EVOLUTION MEDICAL INSTANT COLD PACK DELUXE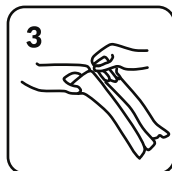

Nur einmal nutzbar – Single use only à usage unique – Usaggio unica

Komfortable, effektive Schmerzlinderung durch das Cold Pack
Confortable, effective pain relief with absorbent pad
Pratique, soulagement efficace de la douleur par le Cold Pack
Comodo, sollievo dal dolore efficace col Cold Pack

Verwendbar für konsistente Kältetherapien
Use for consistent cold therapy
Convient pour thérapie par le froid
Adatto per crioterapia coerente

Therapeutische Kühlbinde zur äusserlichen Anwendung auf der Haut
Safe, therapeutic temperature that can be applied directly to skin
Fixer la température thérapeutique qui est directement applicable sur la peau
Fissare la temperatura terapeutica che è direttamente trasferibile alla pelle

Selbstaktivierendes Cold Pack, kein Einfrieren nötig
Activates instantly, no refrigeration required
Auto-activation du Cold Pack, pas de congélateur requis
Auto Attivazione Cold Pack, richiesto senza congelatore

Steril – Steril – stérile – sterile
Latex frei – Latex free – Sans latex – Senza latex

Inhalte: Urea, Wasser – Contents: Urea, Water

Contenu: Urée, Eau – Contenuto: Urea, Acqua

 Intco Medical (HK) Co., Limited
Flat/rm 19c, Lockhart Centre,
301-307 Lockhart Road, Wan Chai, HongKong

Manufactured for
Evolution Marketing AG,
Bühlstrasse 25
8125 Zollikerberg - CH
CHE-153.253.468

Distributed by
Evolution Medical GmbH CHE-113.517.158

Lot.Nr S01-0001-CH

7 640177 150127 >

EVOLUTION MEDICAL INSTANT COLD PACK DELUXE

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Instructions for Single Use Application

1. Keep the Cold Pack inside the package. Find the activation capsule inside the Cold Pack and press it until it bursts.
2. Shake pad well to mix contents.
3. Remove from plastic pouch and apply to affected area with white stripe facing down. Remove and leave area uncovered for 60 minutes.
4. Dispose into trash after single use.

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Cautions & Warnings:

- Do not puncture or strike the Cold Pack in any manner. Do not use if punctured, torn, or leaking.
- Individuals with circulatory problems should use only at the direction of a physician.
- Do not apply to broken skin. Keep out of reach of children.
- Check patient regularly during product use. Keep away from extreme heat and open flame.
- If accidentally swallowed or into contact with eyes, consult a physician immediately.
- If contents inside pack come in contact to skin, wash area with mild soap and water to avoid irritation.

Storage:

Store at room temperature, Do not refrigerate or freeze prior to use, as frostbite may occur.

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